### **PARALLELE CON BASAMENTO 260212**

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### **PARALLEL BARS WITH PLATFORM: 260212**

It is imperative to follow the instructions below to assure correct assembly, installation as well as correct use of the parallel bars 260212.

Obligation to anchor to the floor, according to din en 12182.

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
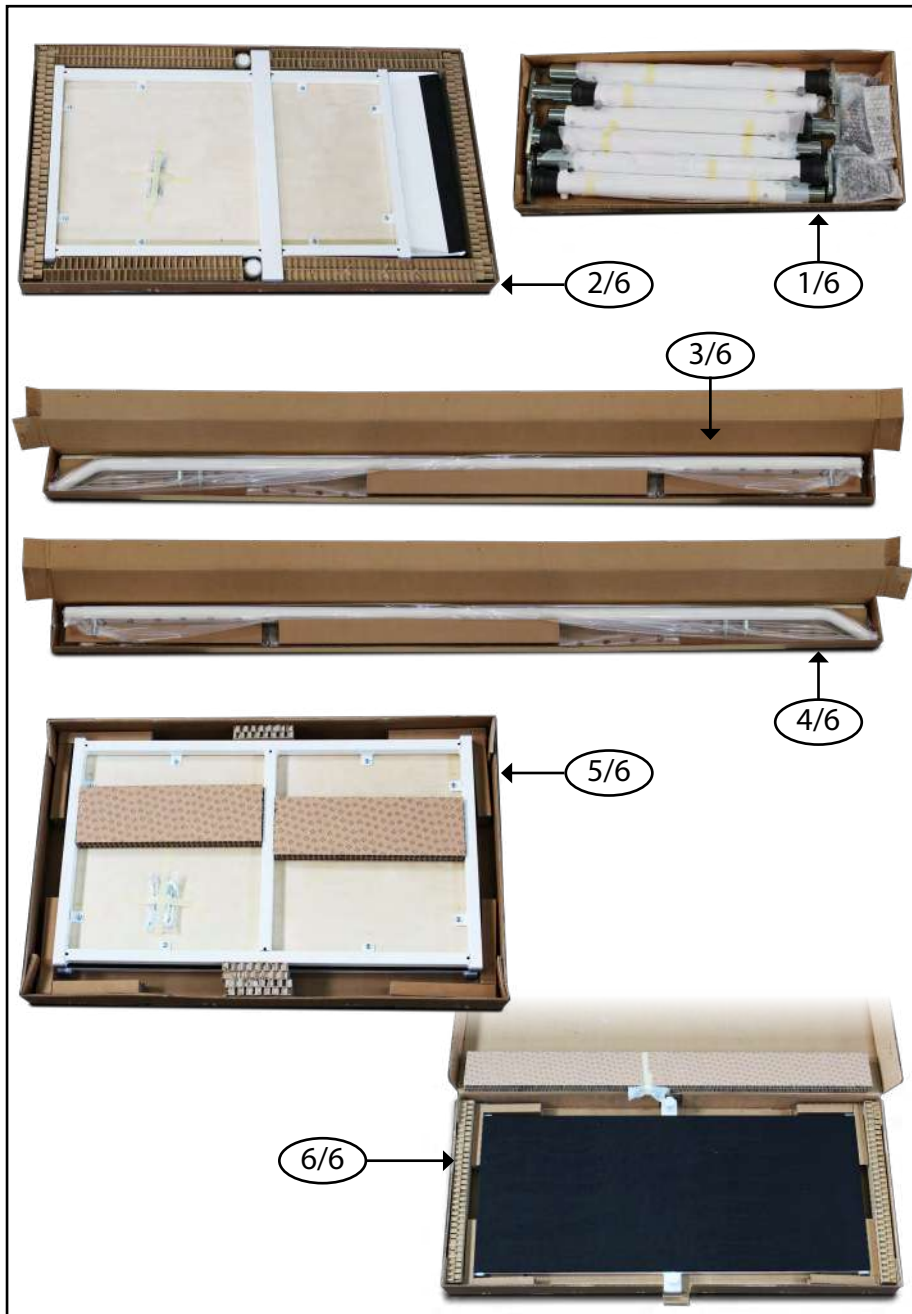
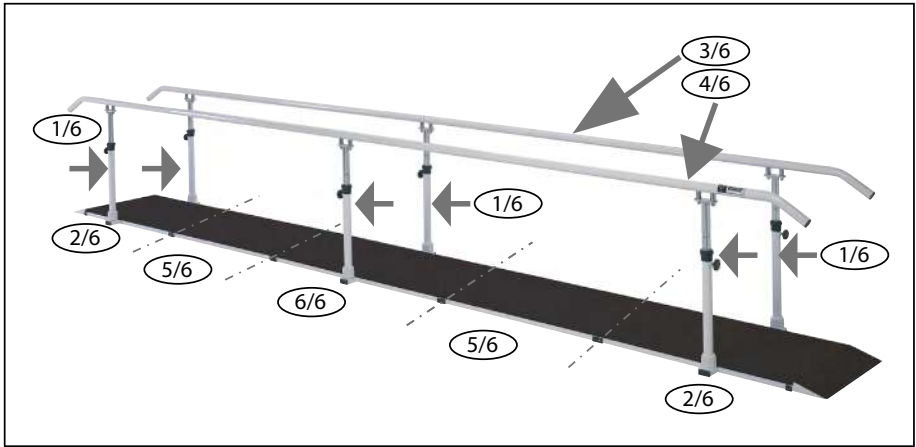
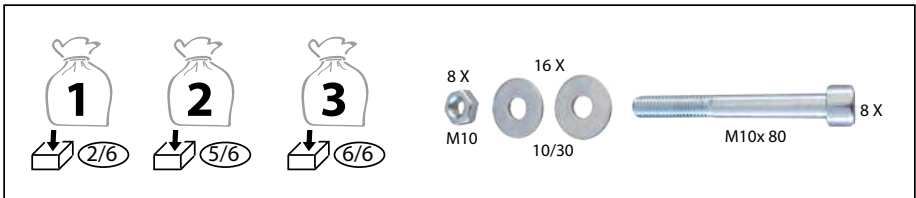
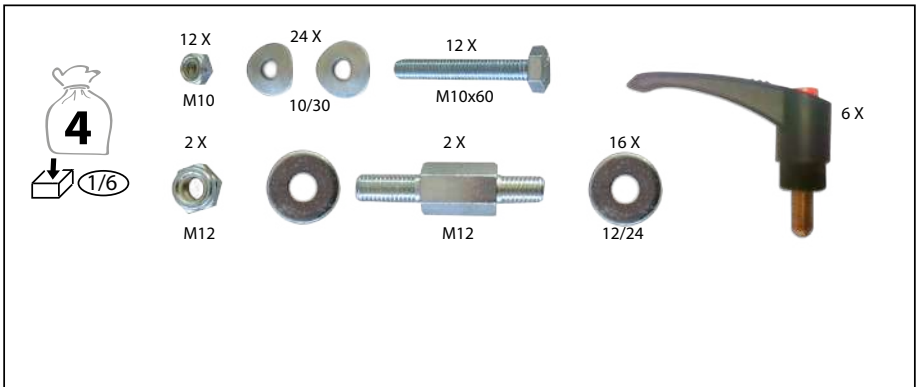
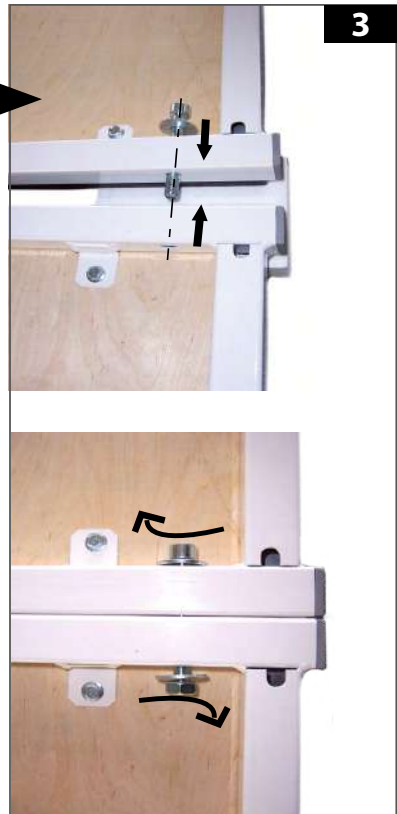
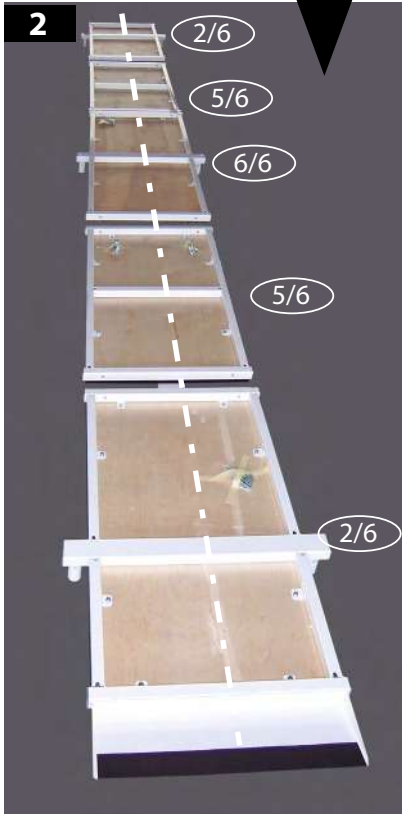
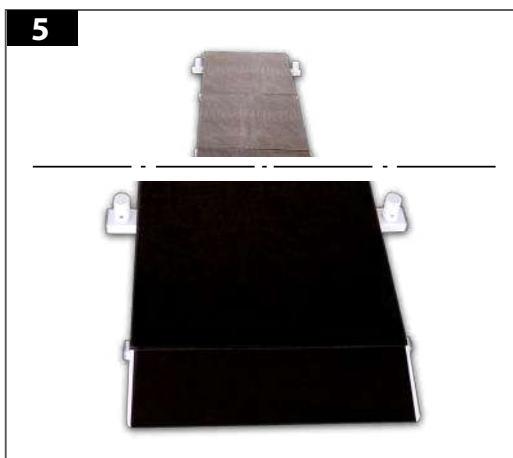
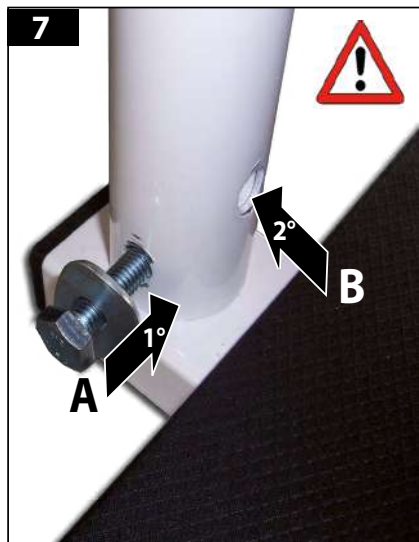
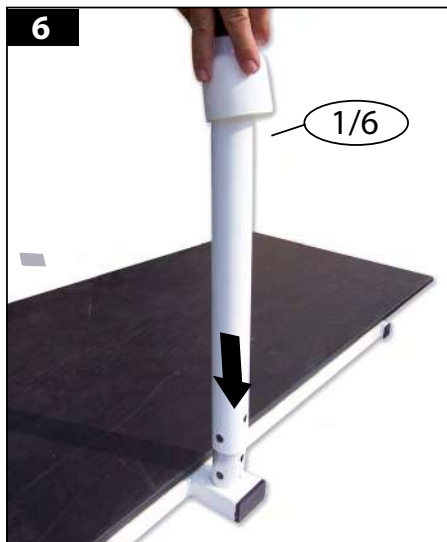
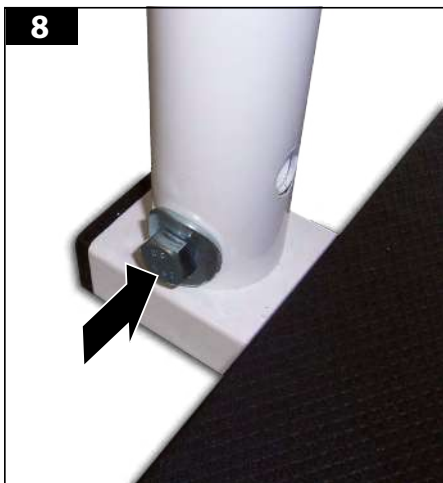
**NOTE: TO ENSURE CORRECT ASSEMBLY OF THE ITEM IT IS NECESSARY TO START AT THE BOTTOM HOLE AND PROCEED FROM THERE UPWARDS**

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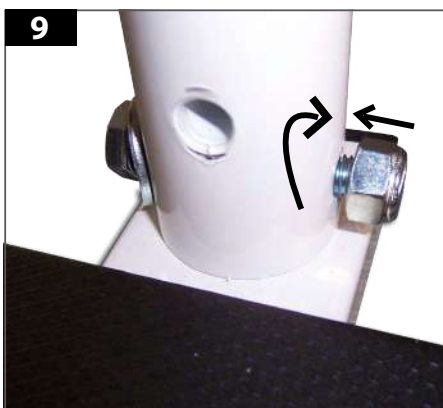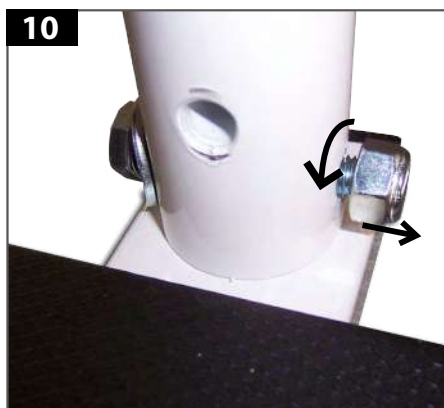
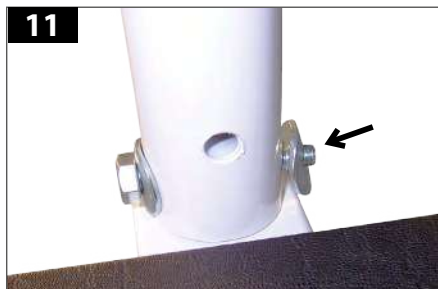
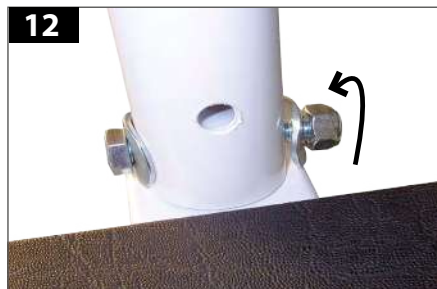
• PROCEED AS DESCRIBED ABOVE TO ANCHOR ALL OTHER WHITE SUPPORT TUBES

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• INSERT ALL SIX SKID PIPES INTO THE SIX SUPPORT TUBES

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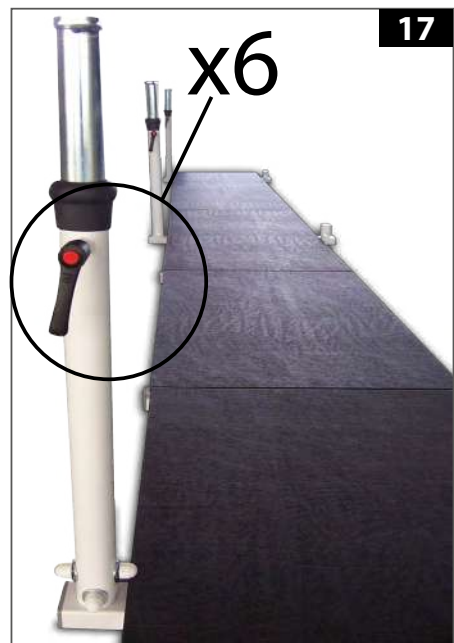
• FIT ONE BOLT LEVER TO THE SIDE OF EACH SUPPORT TUBE

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
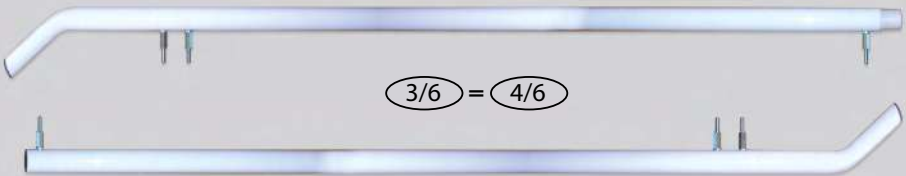
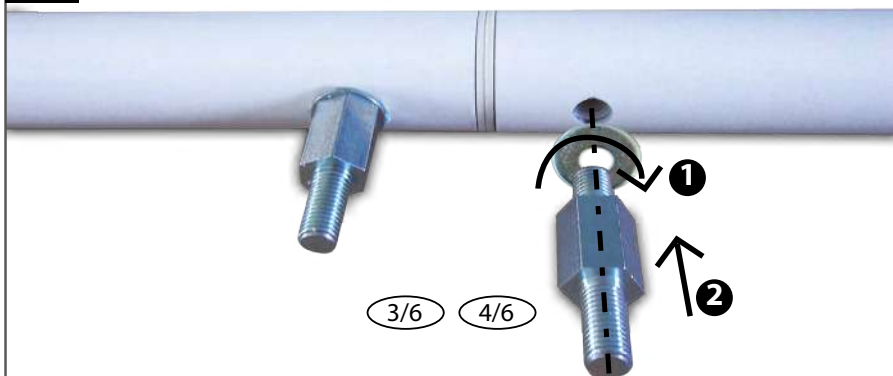
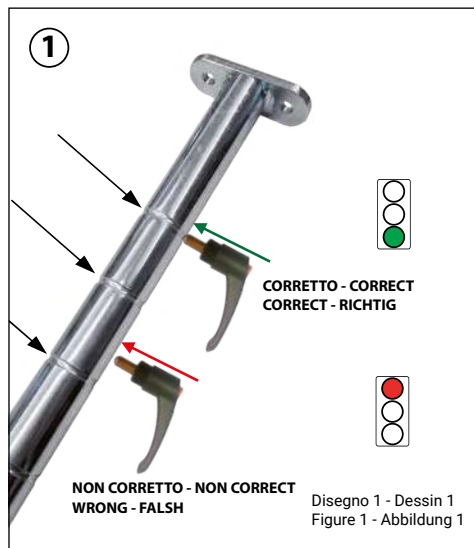
THE HEIGHT OF THE PARALLEL BARS IS CORRECTLY AND SECURELY ADJUSTED, ONCE THE FASTENER TIGHTLY SITS IN ONE OF THE SPECIFICALLY DESIGNED NOTCHES.

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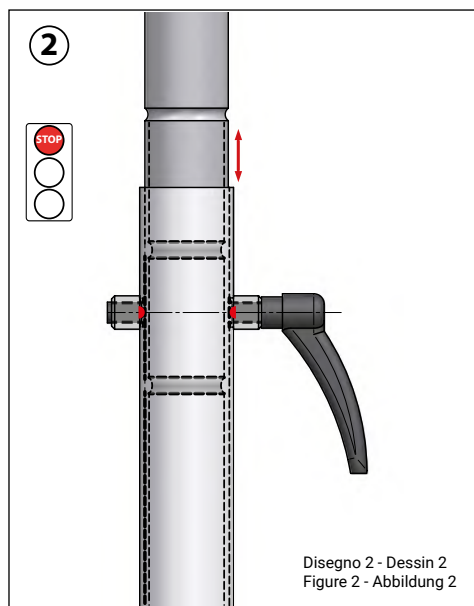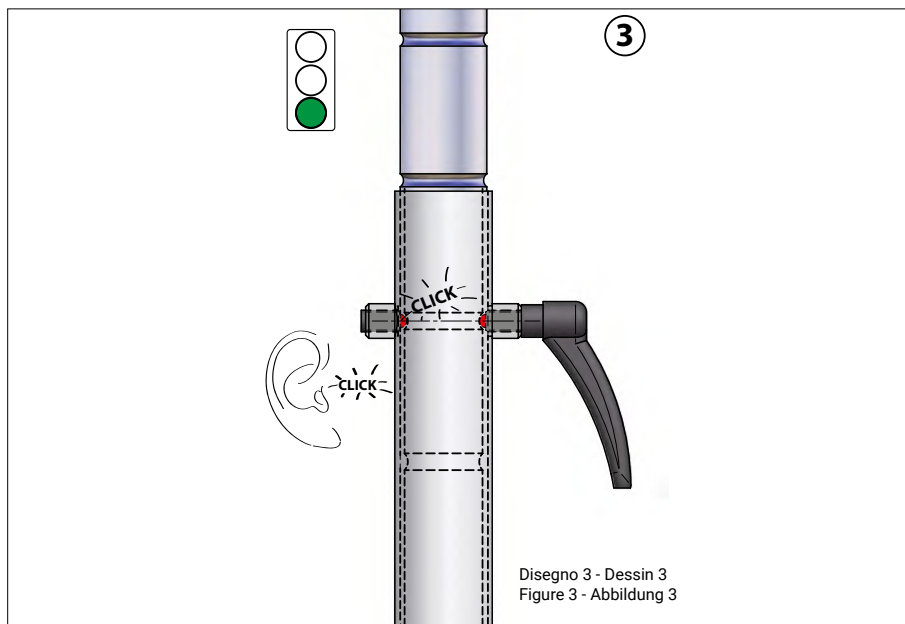
**PAS CORRECTE.**

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THE NOTCH POSITIONING FOR HEIGHT ADJUSTMENT IS: **WRONG.**

REFER TO FIGURE 3 FOR CORRECT ALIGNMENT.

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THE NOTCH POSITIONING FOR HEIGHT ADJUSTMENT IS: **CORRECT.**  
 IN THIS ALIGNMENT THE FASTENER (A) IS CORRECTLY POSITIONED TO SNAP IN  
 THE DESIGNATED NOTCH.

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## CLASSIFICATION AND USE

### PARALLEL BARS WITH PLATFORM: 260212

Read the instructions and use the product as directed according to its function.

Do not use the product in a way other than its intended purpose.

#### INTENDED USE

Parallel bars with base for continuous use in manned hospital environment.  
Immobilised patients with simple fractures or residents in aged-care facilities, who require a firm support, adjustable in height and width for a greater feeling of security.

#### TYPE OF PATIENT

Adults who need to learn how to walk again or simply to move around safely inside a room and along a path.

#### PLACE OF USE

Hospitals, specialised clinics, aged-care facilities

#### OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals, physiotherapists, and related industries.

The instructions given to the patient by the reference healthcare professional suffice.

#### CONTRAINDICATIONS

Follow the safety instructions in user manual.  
Not suitable for sports use.

#### OTHER INDICATIONS

The device does not have essential levels of performance.

**Report all serious incidents that occur due to the device.  
Report the incident to the manufacturer and the competent authority**

**At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.**

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**Weight:** 126 kg

**Storage:** dry, dark place, -20°+60°

**Cleaning:** it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

**Operating temperature:** 0° / +40°

**Expected lifetime:** it is estimated that the device can be used for 10 years.

It is mandatory for the user and the patient to use disposable gloves and shoes with clean rubber soles with disposable overshoes

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