PARALLELE CHIUDIBILI EASY CLICK: 260215, 260217

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FOLDING PARALLEL BARS EASY CLICK: 260215, 260217

It is imperative to follow the instructions below to assure correct assembly, installation as well as correct use of the parallel bars 260215-217. Obligation to anchor to the floor, according to din en 12182.

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 **Read the instructions and use the product as directed according to its function.**

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
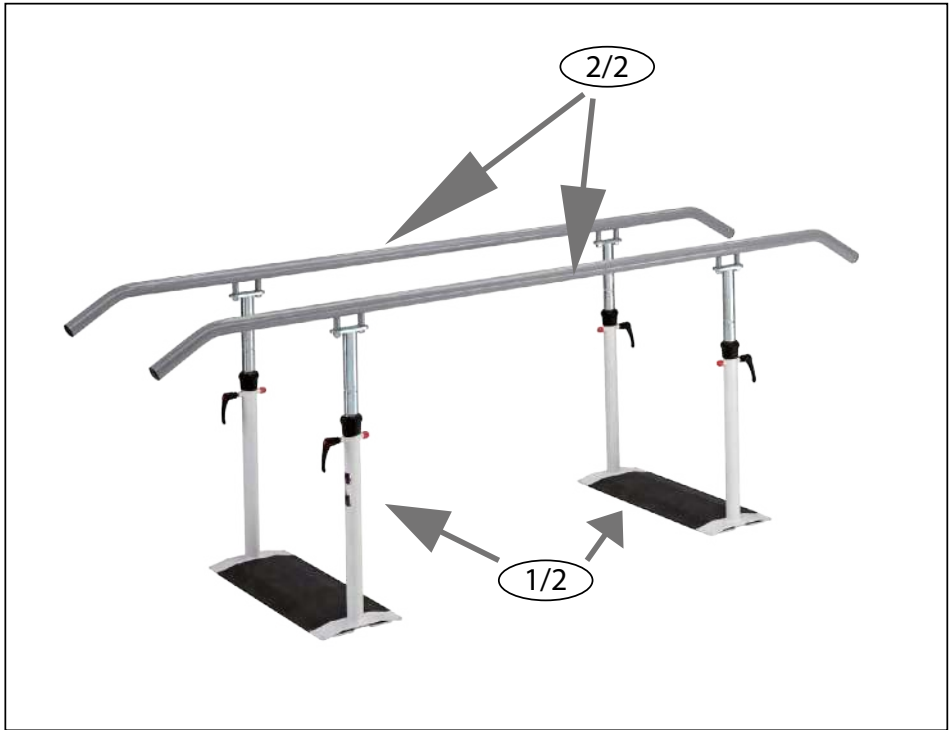
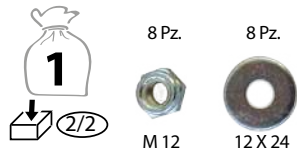
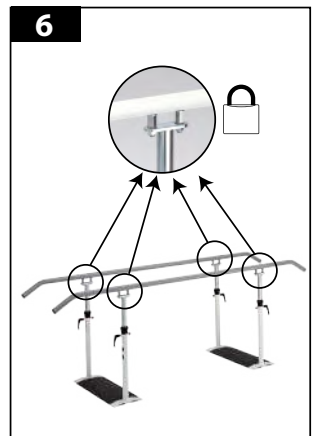
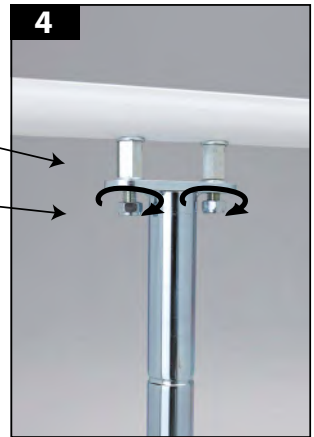
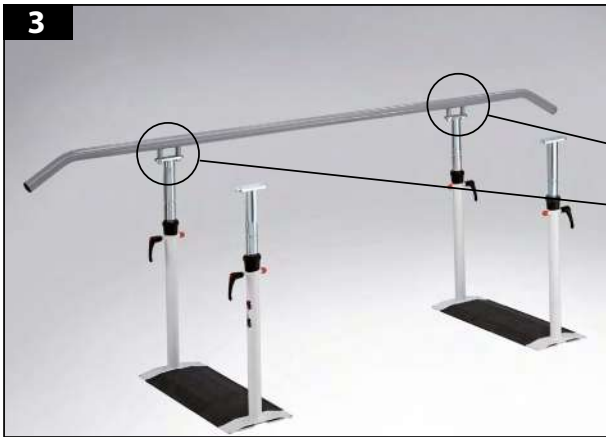
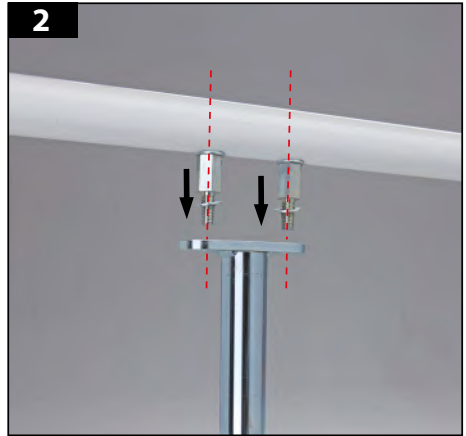
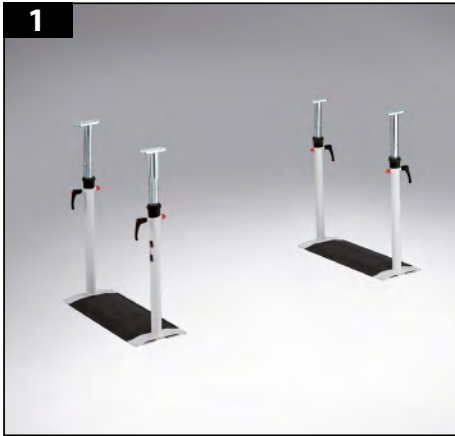
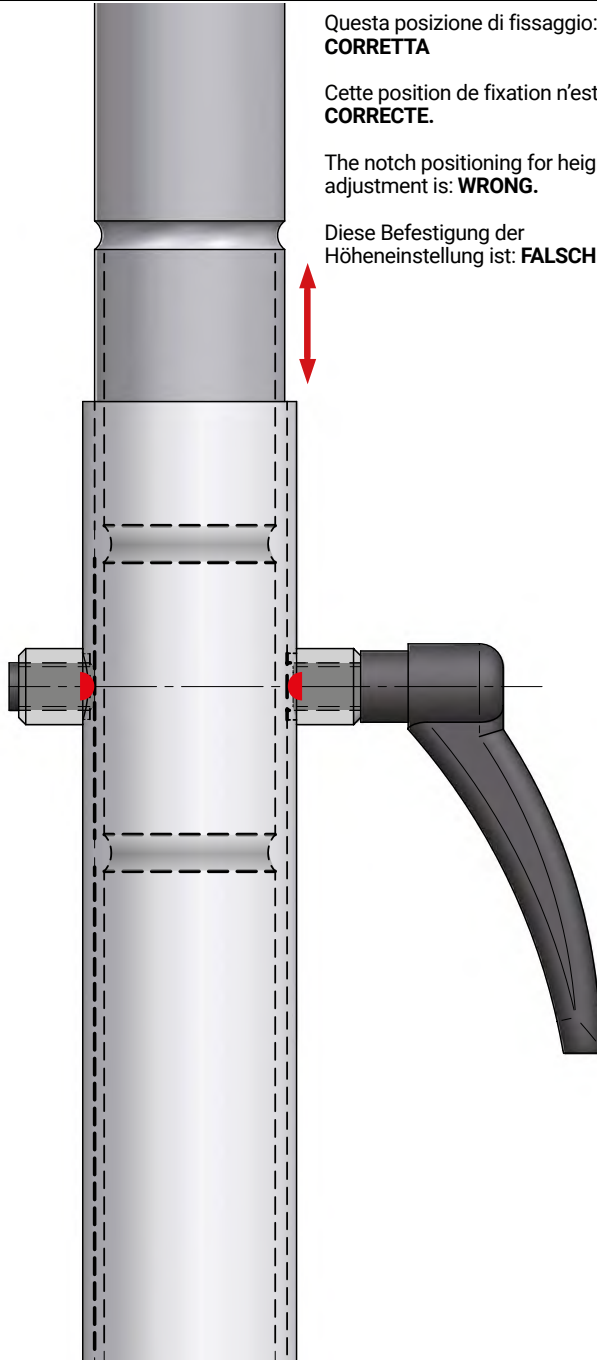
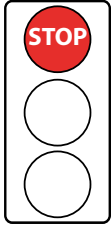
The notch positioning for height adjustment is: **WRONG.**

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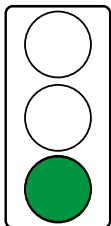The notch positioning for height adjustment is: **CORRECT.**
In this alignment the fastener (A) is correctly positioned to snap in **the designated** notch.

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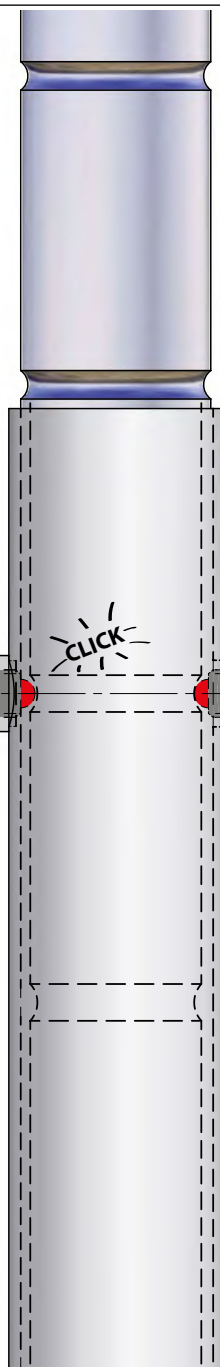
THE HEIGHT OF THE PARALLEL BARS IS CORRECTLY AND SECURELY ADJUSTED, ONCE THE FASTENER TIGHTLY **SITS IN ONE** OF THE SPECIFICALLY DESIGNED NOTCHES.

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Obligation to fix to the floor, as required by UNI EN 12182

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CLASSIFICATION AND USE

FOLDING PARALLEL BARS EASY CLICK: 260215, 260217

Read the instructions and use the product as directed according to its function.

Do not use the product in a way other than its intended purpose.

INTENDED USE

Foldable parallel bars for rehabilitation use. For patients immobilised by simple fractures, motor recovery after minor trauma.

TYPE OF PATIENT

Adults who need short periods of walking rehabilitation.

PLACE OF USE

Medical clinics with limited spaces, home.

OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals, physiotherapists, and related industries.

The instructions given to the patient by the reference healthcare professional suffice.

CONTRAINDICATIONS

Follow the safety instructions in user manual. Not suitable for sports use.

OTHER INDICATIONS

The device does not have essential levels of performance.

Report all serious incidents that occur due to the device. Report the incident to the manufacturer and the competent authority

At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.

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Weight 260215: 48 kg **Weight 260217:** 52 kg

Storage: dry, dark place, -20°+60°

Cleaning: it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

Operating temperature: 0° / +40°

Expected lifetime: it is estimated that the device can be used for 10 years.

It is mandatory for the user and the patient to use disposable gloves and shoes with clean rubber soles with disposable overshoes

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